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*Bruksanvisning/Instructions for use*

# NOVISTA

Bruksanvisning/Instructions for use

SE ..... 4

DK..... 6

NO ..... 8

FI .....10

EN ..... 12

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## [EN] Instructions for use – Fibertyngdvästen

Novista's Fibertyngdvästen is a weighted vest for use by anyone suffering from anxiety and anguish, and for many wearers it becomes an important part of getting through daily life. Novista's weighted vest can be used to successfully alleviate anxiety affecting both mood and motor control as well as diagnoses such as autism, ADHD, mental illness and more. Novista's weighted vest is suitable for use at school, home or work.

Adults	XS	S	M	L	XL	XXL
Chest cm	76-84	84-94	94-102	102-110	110-118	118-126
Waist cm	68-80	80-88	88-96	96-104	104-112	112-120

Children	110/116	122/128	134/140	146/152
Chest cm	52-58	58-64	64-70	70-76
Waist cm	52-55	55-58	58-61	61-64

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Novista's Fibertyngdvästen is a weighted vest with a firm, integrated girdle with a velcro fastener that is tightened at the waist to suit the wearer's preferences. This enables the wearer to adjust the feeling of weight and pressure it exerts. Novista's weighted vest has been developed in collaboration with psychiatrists and stress therapists and is classed as a medical device. It does not attract attention as it looks just like a normal vest.

## How Novista's weighted vest works

The vest hugs the body, rather like a warm embrace. This affects the body's sense of position and movement (proprioception) and can help the wearer to feel a comforting calm and greater ease concentrating.

- Helps to prevent anxiety affecting motor control
- Can reduce anguish and stress
- Can increase sense of security
- Aids focus

## Safety instructions

Do not use Novista's weighted vest in environments or ways that entail a risk of fastening during movement. Consult an expert if you have impaired balance, breathing difficulties or any other condition that affects your ability to wear the weighted vest in a safe manner, whether such condition is permanent or a temporary symptom of injury or illness.

If you have any questions or thoughts about risks, please feel free to contact our customer service department by e-mailing [info@novista.se](mailto:info@novista.se)

## Materials and care instructions

Novista's weighted vest is made from a dirt and stain resistant material. This means that the vest can be wiped clean with a damp cloth and hung up to air rather than washed.

If the vest is dirty enough to need washing, it is important that the zipper is left open and the velcro girdle is completely closed. Use a washing bag if possible.

Novista's weighted vest can be washed at up to 60 degrees, although with consideration for the environment we recommend that the vest be washed at a maximum of 40 degrees.

Material	Polyester
CE marked	Yes
Guarantee	2 years*

\* Not valid if handled carelessly or without following the care instructions.

# NOVISTA

Novista of Sweden AB • Johannebergsvägen 6 • SE-312 72 Skummeslövsstrand  
info@novista.se • +46 (0)431-49 13 10  
[www.novista.se](http://www.novista.se)